

making life


easy

Shower Stool

User Manual



User Manual

 Please read instructions before use.

Assembly Instructions

Carefully remove the shower frame and components from the box.

Examine all parts for damage. If any parts are missing or damaged please call 1300 363 303. DO NOT use the shower stool until all parts are received.

Components



Seat x 1



Leg Frame x 2



Screws x 5
Washers x 5



Armrests x 2




Legs x 4




Step 1:

Place seat bottom upside down on a flat surface. Attach the leg frames to the seat and fasten with the screw and washer in the center. Ensure all connections are securely tightened.

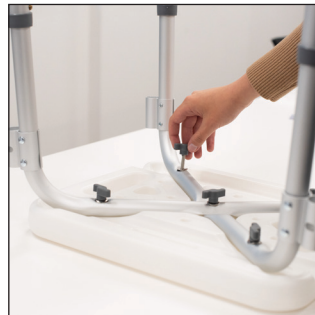

Step 2:

Attach the extension legs to the leg frame. Adjust the height by aligning the spring buttons with the adjustment holes on each leg. Ensure the spring buttons are fully engaged to securely lock each leg in place. Ensure all legs are adjusted to the same height.

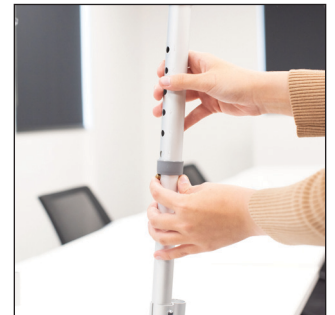

Step 3:

Position the chair upright. Insert the armrests on each side by pressing the spring button inwards. Ensure the spring buttons are facing outwards and are fully locked in place.

Safety and Maintenance



Read this manual before using the Shower Stool.

Always ensure the chair is placed on a flat, stable surface before use. Adhering to the maximum user weight specified by the manufacturer is crucial, as exceeding this limit can compromise the chair's stability and lead to injury or damage. The shower stool is designed for single-user use only and should not be used as a step stool or for other unintended purposes.

Avoid using the chair on uneven surfaces and refrain from exposing it to extreme conditions, such as high-pressure water jets, that may cause instability or rust. If obstacles or uneven surfaces are present, approach these areas slowly and cautiously. The chair should not be used on steep inclines, and the maximum safe angle for use is typically 10% unless stated otherwise by the manufacturer.

Regularly ensure that all screws, bolts, and locking mechanisms are secure, and do not use the chair if any parts are loose or damaged.

Maintaining the chair involves routine cleaning after every use. Wipe it down with warm water and mild soap, rinse thoroughly, and dry completely to prevent mold, mildew, or rust. Inspect the chair frequently for cracks, corrosion, or loose parts, and replace damaged components such as rubber tips, screws, or worn-out seats to maintain safety. Metal components should be dried thoroughly after each use to prevent rust, and the chair should not be left in damp environments for extended periods.

Regularly check the condition of the legs, seat, and backrest for wear and tear, and if wheels or hinges are part of the chair, ensure they are clean, functional, and not excessively worn. Damaged parts should be replaced promptly. When not in use, store the chair in a dry, ventilated space.

By adhering to these safety and maintenance practices, you can ensure your shower stool remains reliable, prolong its lifespan, and reduce the risk of accidents.

Please contact your place of purchase to arrange the purchase of any spare parts you need.

Note: This shower stool is designed for limited or domestic usage and is not designed for high or commercial usage.

Specification

Seat Width 460 mm	Overall Width 510 mm
Seat Height 400-500 mm	Max User Weight 114 kg
Seat Depth 330 mm	Warranty 12 months
Unit Weight 2.4 kg	

Product Warranty

Goods with warranty term

- Goods have a 12 month warranty from the date of purchase.
- Valid proof of purchase is required for all warranty claims.
- The warranty does not extend to any subsequent purchaser or owner of the goods.
- The warranty covers manufacturing faults only, and does not cover change of mind or damage, fault or failure due to external causes, including (but not limited to) accident, abuse, misuse, incorrect assembly or normal wear and tear.
- Any unauthorised repairs, alterations or modifications or any attempt to open or dismantle the good(s) will render the warranty void.
- Defective goods, together with valid proof of purchase, should be returned to the place of purchase or shipped directly to the manufacturer. If shipped directly to the manufacturer, you must prepay the shipping and transportation charges and insure the shipment or accept the risk of loss or damage during shipment and transportation.
- The manufacturer will determine at no cost to you the cause of the fault (if any) within a reasonable time frame. Any repaired or replaced goods, original goods determined not faulty, or refund (if a major failure is determined) will be sent to the pharmacy you returned the goods to for collection, or shipped to you freight prepaid if you shipped the goods directly to the manufacturer.
- Our goods come with guarantees that cannot be excluded under Australian Consumer Law. You are entitled to a replacement or refund for a major failure and you are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. Goods presented for repair may be replaced by refurbished goods of the same type rather than being repaired.

making life
easy

Ph: 1300 363 303
www.makinglifeeasy.com.au
Level 8 637 Flinders Street
Docklands VIC 3008