

making life


easy

Raised Toilet Seat with Lid 100 mm

User Manual



Assembly Instructions

 Please read instructions before use.

The Raised Toilet Seat has been designed to accommodate a wide variety of toilet bowl types. However, certain bowl types will require the removal of the existing seat and/or lid to ensure proper fit when installed.



Step 1:

Lift up toilet seat and lid. You may also need to remove the entire seat and lid from the toilet (varies depending on the size of the toilet).



Step 2:

Make sure the toilet seat is clean and sanitised.

Step 3:

Add adjustment clamps into the toilet seat raiser.

Place the raised toilet seat onto the toilet. Make sure the rear anchor of the raised toilet seat sits beneath the back of the rim.



Step 4:

Ensure the front lip of the toilet seat raiser is positioned against the front of the toilet rim.



Step 5:


Turn the adjustment clamp clockwise until the raised toilet seat is securely locked on the toilet. Do not over tighten the clamps.



Step 6:

Pull on the raised toilet seat to make sure it is securely locked on the toilet. When the raised toilet seat is properly installed, the unit will sit flat on the top of the bowl and should fit securely against the inner rim of the bowl.

Safety and Maintenance

 Please read this manual before using the Raised Toilet Seat.

- Always be sure that the seat is correctly and securely locked in place before using.
- Be sure that the adjustment knob on side of the seat is tight at all times, thus securing the seat to the toilet bowl.
- Check regularly to make sure the raised toilet seat is securely locked onto the toilet.
- Clean regularly with a toilet sanitiser or with regular household cleaning products. DO NOT use abrasive powders or scouring pads.